

**DO NOT REMOVE
FROM EQUIPMENT**



Millbrook
HEALTHCARE GROUP

Generic Electric Hoist – Service User and Carer Daily Checks

Remember: Checks are mandatory

CONTACT
+44(0) 8448111156

JOERNS HEALTHCARE LIMITED
WORCESTER ROAD
DRAKES BROUGHTON WR10 2AQ

CE

MODEL:
OXF – MAJORE – UK
SWL 200 KGS

Mfg Date: 18.07.2013

SN 23100000280

EXAMPLE

1. Check Safe Working Load (SWL) is clearly displayed on the specification label.

MILLBROOK HEALTHCARE
LOLER INSPECTION

S/N: 23100000280

Date: 18 / 05 / 13

Tech: M1

Next Inspection due:
Date: 18 / 11 / 13

21261930
CES200
S/N: 2310000
When No Lo
Please T

EXAMPLE

2. Check the last LOLER inspection is within the last six months.



3. If SWL is not displayed or LOLER inspection is over six months old, DO NOT USE. Arrange inspection.



4. Check that hoist moves freely on its castors. Check the breaks on the castors work correctly.




5. Check that spreader bar rotates and swivels freely.



6. Ensure the spreader bar is firmly attached to the boom; and the boom is not damaged (bent or dented).



7. Check sling hooks and side suspenders for excessive wear.

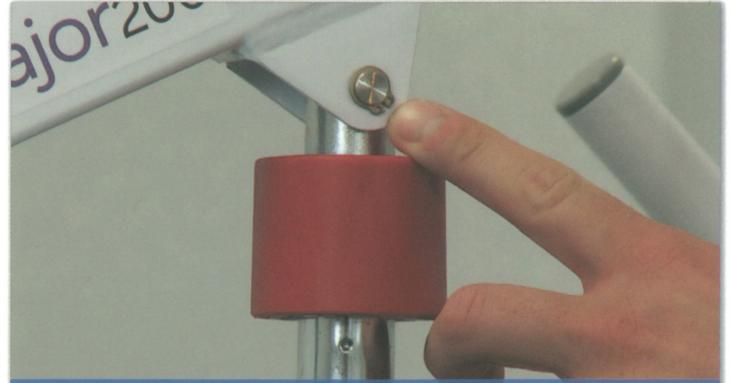

8. Check all fastenings on the actuator. Check the actuator for signs of damage.



9. Ensure legs open and close correctly.



10. Use hand control to check if boom raises and lowers correctly.



11. Ensure emergency stop works correctly.



12. Ensure the mast stem fully engaged into the base. The locking knob fully tightened.



Millbrook
HEALTHCARE GROUP

If in doubt, please call your local Millbrook Contact Centre.